



Smartmold

Indication

Smartmold is indicated for the anatomical correction of facial deformities, filling of tissues and harmonization of the facial structure; it is a durable and definitive solution.

Implantation procedures

Step 1: Remove the biomodel and Smartmold parts from the packaging.

Step 2: Test the fit of the Smartmold by checking the adaptation of the PMMA part to the biomodel, checking the alignment of the perforations.

Step 3: Install the Smartmold in the indicated anatomical location, securing it with screws.



Cautions and Warnings

- ✓ SINGLE USE product. DO NOT REUSE OR REPROCESS.
- ✓ The Smartmold system must be implanted by a qualified professional, in a surgical environment and with properly sterilized instruments.
- ✓ Do not use the product if the packaging is damaged or the seal is broken.
 - ✓ Do not use the product if found damaged.
 - ✓ Do not use the product after the expiration date.
 - ✓ Do not use if the model/reference shows any discrepancies.

Adverse Events

Every surgical procedure presents risks and the possibility of complications. Adverse events may include:

- Loosening, displacement or fracture of components.
- Feeling of pain, discomfort or annoyance.
- Edema, bruising, inflammation, infection, exfoliation, abscess formation, hyperplasia, and complications associated with anesthesia.



Follow us on
social media to
learn more

#muchmorethananimplant

+55 (61) 98151 1637
cpmhdigital.com